## Protac KneedMe<sup>®</sup>

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GB - Directions for use

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SE - Bruksanvisning

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Medical Device

[www.protac.dk](http://www.protac.dk)

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## Protac KneedMe®

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# GB - Directions for use

## Protac KneedMe®

You have chosen a Protac quality product. Protac guarantees that you have purchased an original Danish designed product developed in partnership with professionals.

Since 1994, Protac has developed, manufactured and sold sensory stimulating aids to the healthcare sectors in Denmark, Europe, Australia and New Zealand.

Protac products are developed from the theory of sensory integration and are well documented with respect to research, documentation and case descriptions.

Protac production process focuses on environment and safety. Protac plastic balls are certified STANDARD 100 by OEKO-TEX® nr. 2076-310 DTI.

GB

For more details, go to [www.protac.dk](http://www.protac.dk)

## Labelling

The products are labeled in accordance with Regulation (EU) 2017/745 Class I medical devices. CE- and carelabels with information about the manufacturer, raw materials, washing instructions and LOT number are placed inside the cover.

Key to label symbols: please refer to Key to symbols

## General precautions - Safety

Please note that the Protac KneedMe® is not intended as a restriction of personal freedom or as a safety device.

- ⚠ • If a user's movements or impaired cognitive abilities accidentally cause the blanket to slip to the floor, we recommend keeping the user under supervision.
- ⚠ • Take precautions to avoid pressure sores when using Protac KneedMe®.
- ⚠ • Cease using the product if there are any signs of redness or irritation on the user's thighs or buttocks as a result of using Protac KneedMe®.
- ⚠ • Not suitable for use on children.
- ⚠ • When using Protac KneedMe® in a car, it must not be used by the person driving the car.
- ⚠ • If a passenger is using Protac KneedMe®, it must lie loosely over the knees
- ⚠ • Must not be used near an open flame.

## Incidents

- Incidents related to the use of Protac KneedMe® must be reported to the manufacturer
- Serious incidents related to the use of Protac KneedMe® be reported to the manufacturer and the competent authority of the country where the incident takes place.

## Product purpose and areas of use

Sensory integration is the idea behind the development of all Protac products. The theory originated in the 1970s in the USA, and was developed by A. Jean Ayres, an occupational therapist and psychologist. The theory is based on how the brain processes sensory impressions, and how the various sensory stimuli can be influenced via applying weight and pressure to the tactile sense (the sense of touch) in the skin and the proprioceptive sense (the sense of movement) in muscles and joints. The pressure, movement and weight of the balls act on these senses in particular.

Protac KneedMe® has been developed as an aid for people with disabilities. Adults and the elderly can use Protac KneedMe®:

- Brain damage and other neurological diseases.
- ADHD and Autism
- Psychiatric diseases
- Developmental disorders, sensory disorders.
- Physical and mental unrest.
- Concentration problems and learning difficulties.

GB

Protac KneedMe® is used by people with brain injuries and other neurological diseases. These patients often suffer from neurogenic disorders. The movements and pressure of the balls give the body sensory stimuli that contribute to an increased body sensation.

For people with ADHD and autism, Protac KneedMe® is used to create calm and concentration. The movements and pressure of the balls on the body provide sensory stimuli that create calmness and help to define the body. The use of the knee blanket enables users to maintain concentration for longer periods.

Protac KneedMe® is used for people with psychiatric illnesses. Many have violent movements and psychotic behavior which subsides when they get the knee blanket over them. Patients become more present in contact with others.

Protac KneedMe® is used for people with eg developmental disorders, sensory disorders to create increased sensory stimulation for the body.

Protac KneedMe® is used for people with physical and mental unrest, such as restlessness in the legs. The movement of the balls in the knee blanket provides sensory stimuli in the legs and an increased body sensation and calmness

Protac KneedMe® is used for people with concentration problems and learning difficulties. The balls in the knee blanket provide sensory stimuli which contribute to an increased body sensation.

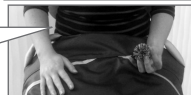
## Using the product

The Protac KneedMe® is laid over the user's lap.

Non slip grip sheet on reverse side.

A carabiner hook is sewn into the top of the pocket for objects such as keys which the user likes to touch.

GB

## Product service life

The product has an expected service life of 5 years.

This is based on correct use, cleaning, washing, service and maintenance.

Products used beyond the aforementioned service life should be assessed for further use.

Responsibility for using the product beyond the stated service life rests with the user.

## Cleaning

Protac KneedMe® is washable. The regularity and need for washing depends on use.

When washing Protac KneedMe®, follow the instructions below.

- Protac KneedMe® can be machine washed in a min. 7 kg capacity washing machine. Use a detergent with no bleaching agents and perfume - see the washing instructions in the cover.
- The cover and ball bag must be washed separately. The laundry bag supplied is recommended for washing the ball bag.

To remove spots using a spot remover, always try first on an area which is not visible to test for damage or discolouring. Protac does not recommend the use of spot remover.

Protac cannot be held liable for incorrect washing and maintenance.

## Maintenance

We recommend airing the knee blanket regularly.

Check the Protac KneedMe® every time it is used for defects and deficiencies, and inspect the seams, zip, fabric, balls and general condition once a year. If any serious defects are found, withdraw the blanket from use until repairs have been made and it is safe to use again.

## Transport and storage

The Protac KneedMe® is supplied in a handy carrying case.  
If the blanket is not in use, store it carefully in its bag.

## Disposal

The Protac KneedMe® can be broken down into the following components for disposal:  
Plastic and textile.

- The plastic balls can be disposed of as hard plastic
- All parts can be disposed of as combustible waste.
- Waste must be disposed of in accordance with applicable rules.

GB

## Guarantee

2 year guarantee against manufacturing defects.

### The guarantee will be invalid in the event of the following:

- over-use
- misuse
- non-maintenance or incorrect maintenance

**Traceability:** Production month, year and LOT number are marked on the washing label.  
These must not be removed from the product.

## Key to symbols

Example only. Always follow the recommended care instructions on the label found inside the product.

Washing bowl: Maximum washing temperature (see care label)

Warning:  
See under Safety

Triangle:  
Do not bleach

User information

Square with a circle:  
tumble drying possible  
(see care label)

For indoor use

CE marked according to  
(EU) 2017/745 Class I  
medical devices

Iron:  
Do not iron

Manufacturer

Circle:  
Do not dry clean

LOT number

## Technical specifications

**Standards:** Comply with general requirements for aids acc. to the following harmonized standards:

**EN ISO 13485:2016/A11:2021:** Medical Devices – Quality management systems

**EN ISO 14971:2019/A11:2021:** Medical devices - Application of risk management to medical devices

**DS/EN ISO 21856:2022 :** Medical devices - Assistive products - General requirements and test methods.

## Service information

No other balls can be used in a Protac KneedMe® blanket than those used by the manufacturer.

GB

## Guide

- to daily inspection of the Protact KneedMe® before use

- 1 • All parts are in place, no loose parts
  
- 2 • No sign of holes, damage to the textile or seams

Protac A/S  
Niels Bohrs Vej 31 D, Stilling  
DK-8660 Skanderborg  
Tel.: +45 86 19 41 03  
e-mail: protac@protac.dk

Distributor list: Please refer to: [www.protac.dk/kontakt](http://www.protac.dk/kontakt)

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