Protac MyFit®

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GB - Directions for use

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SE - Bruksanvisning

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Medical device

www.protac.dk

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Protac MyFit®

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GB - Directions for use

Protac MyFit®

You have selected a quality Protac product. Protac guarantees that you have purchased an original, Danish designed product developed in consultation with experts.

Since 1994, Protac has developed, manufactured and sold sensory stimulating aids to the healthcare sectors in Denmark, Europe, Australia and New Zealand.

Protac products are developed on the basis of theories on sensory integration, and are supported by extensive research, documentation and case studies.

Protac focuses strongly on protection of the environment and safety during its production process. Protac plastic balls are certified according to STANDARD 100 by OEKO-TEX® no. 2076-310 DTI.

For more details, go to www.protac.dk

GB

Labelling

The products are labeled in accordance with Regulation (EU) 2017/745 Class I medical devices. The label with details of manufacturer, raw materials and washing instructions is sewn into the sideseam inside the MyFit

Key to label symbols: please refer to Key to symbols

General precautions - Safety

- ⚠ • Please be aware of how the user reacts.
- ⚠ • The Protac MyFit® vest must not be used as a swimming aid or life vest.
- ⚠ • Not for unsupervised use by users with a poor sense of balance.
- ⚠ • Not for unsupervised use by users unable to care for themselves.
- ⚠ • Do not use near open fire.
- ⚠ • Must not be used while sleeping.
- ⚠ • Must not be used by driver nor passengers when driving in a car
- ⚠ • The vest must be removed before the Heimlich method is performed

Incidents

- Incidents related to the use of Protac MyFit® must be reported to the manufacturer
- Serious incidents related to the use of Protac MyFit® must be reported to the manufacturer and the competent authority of the country where the incident takes place.

Product purpose and areas of use

Sensory integration is the idea behind the development of all Protac products. The theory originated in the 1970s in the USA, and was developed by A. Jean Ayres, an occupational therapist and psychologist. The theory is based on how the brain processes sensory impressions, and how the various sensory stimuli can be influenced via applying weight and pressure to the tactile sense (the sense of touch) in the skin and the proprioceptive sense (the sense of movement) in muscles and joints. The pressure, movement and weight of the balls act on these senses in particular.

The Protac MyFit® was developed to help sufferers of function impairment. Children, young people, adults and the elderly can benefit from the vest:

- Brain damage and other neurological diseases.
- ADHD and Autism
- Psychiatric diseases.
- Developmental disorders, sensory disorders.
- Physical and Psychological unrest.
- Concentration problems and learning disorders.

The Protac MyFit® is used for treatment of the after-effects suffered by brain damage patients, to strengthen balance and body awareness.

It is used for ADHD and autism sufferers to help with concentration and learning difficulties, as the thickness and pressure of the balls give extra sensory stimuli and better body awareness.

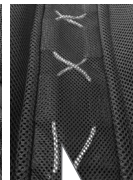
Psychiatric patients often suffer from muscular and psychological unrest. The Protac MyFit® vest is used to create calm and comfort for the user. The weight and pressure of the balls also give a better feeling of body awareness and enhanced wellness.

The Protac MyFit® is used for people with e.g. developmental disorders, sensory disorders to create increased sensory stimuli for the body.

It is used for people with physical and psychological unrest to provide calmness, comfort and better body awareness.

GB

Using the product

<p>1. Put on and zip up the Protac MyFit® vest.</p>		<p>2. Model 800-850-10 can be opened at the shoulders for ease of use.</p>		
<p>3. The vest has a slit on each side, making it comfortable to sit down.</p>		<p>4. Can be adjusted at both sides and at the back.</p>		<p>5. As a safety precaution, the adjustable side and back draw strings are situated in concealed pockets.</p>

Product service life

The product has an expected service life of 2 years.

This is based on correct use, cleaning, washing, service and maintenance.

Products used beyond the aforementioned service life should be assessed for further use.

Responsibility for using the product beyond the stated service life rests with the user.

Cleaning

The Protac MyFit® can be washed. The regularity and need for washing depends on use. See below for washing and cleaning instructions.

- The vest can be washed in a min. 7 kg capacity washing machine with washing detergent with no bleaching agents and perfume. Fabric softener may not be used. The enclosed wash bag must be used. Spinning at low speed. - see also the washing instructions in the vest.

To remove spots using a spot remover, always try first on an area which is not visible to test for damage or discolouring. Protac does not recommend the use of spot remover.

Protac cannot be held liable for incorrect washing and maintenance.

Maintenance

We recommend airing the vest regularly.

Transport and storage

The Protac MyFit® is delivered in a carrying case.

If the Protac MyFit® is not in use, store it securely in the carrying case.

Disposal

Protac MyFit® can be divided into the following components for waste disposal purposes:
Plastic and fabric.

The plastic balls can be disposed of as hard plastic.

All parts of the vest can be disposed of as combustible waste.

Waste must be disposed of in accordance with applicable rules.

Guarantee

2 year guarantee against manufacturing defects.

The guarantee will be invalid in the event of the following:

- Over-use
- Misuse
- Non-maintenance or incorrect maintenance

Traceability: Production month, year and batch number are marked on the washing label.

Key to symbols

Example only. Always follow the recommended care instructions on the label found in the product

Washing bowl: Maximum washing temperature (see care label)

Warning:
See under Safety

Triangle:
Do not bleach

User information

Square with a circle:
tumble drying possible
(see care label)

For indoor use

Iron:
Do not iron

CE marked according to
(EU) 2017/745 Class I
medical devices

Circle:
Do not dry clean

Manufacturer

Batch number

GB

Technical specifications

Standards: Comply with general requirements for aids acc. to the following harmonized standards:

EN ISO 13485:2016/A11:2021: Medical Devices – Quality management systems

EN ISO 14971:2019/A11:2021: Medical devices - Application of risk management to medical devices

DS/EN ISO 21856:2022 : Medical devices - Assistive products - General requirements and test methods.

Service information

No other balls can be used in a Protac MyFit® than those used by the manufacturer.

Guide

- to daily inspection of the Protact MyFit® before use

- 1 • All parts are in place, no loose parts

GB

- 2 • No sign of holes, damage to the textile or seams

Protac A/S
Niels Bohrs Vej 31 D, Stilling
DK-8660 Skanderborg
Denmark
Tel.: +45 86 19 41 03
e-mail: protac@protac.dk

Distributor list: Please refer to: www.protac.dk/kontakt

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