Protac SenSit®

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GB - Directions for use

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SE - Bruksanvisning

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Medical Device

www.protac.dk

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Protac SenSit® Nature

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GB - Directions for use

Protac SenSit® Nature

You have chosen a Protac quality product. Protac guarantees that you have purchased an original Danish designed product developed in partnership with professionals.

Since 1994, Protac has developed, manufactured and sold sensory stimulating aids to the healthcare sectors in Denmark, Europe, Australia and New Zealand.

Protac products are developed from the theory of sensory integration and are well documented with respect to research, documentation and case descriptions.

Protac production process focuses on environment and safety. The textiles are certified according to STANDARD 100 by OEKO-TEX® nr. 1407028 Centexbel. Protac plastic balls are Standard 100 by OEKO-TEX® nr. 2076-310 DTI.

For more information, go to www.protac.dk

GB

Labelling

The products are labeled in accordance with Regulation (EU) 2017/745 Class I medical devices. CE- and care labels with information about the manufacturer, raw materials and washing instructions are placed inside in the cover of the chair.

Symbol explanation on label: see the Key to symbols section

General precautions - Safety

- ⚠ • Please pay attention to the reaction of the user, if he or she does not like being encased in the chair's neck and side wings.
- ⚠ • Make sure that the user does not overheat if he/she is embraced by the chair's neck and side wings.
- ⚠ • Users with poor breathing must use the side and neck wings with care and never unsupervised.
- ⚠ • The chair **must not** be used unsupervised with individuals who require special care.
- ⚠ • Check to see whether any balls might be irritating the user. This is particularly important if the user is incapable of independent movement.
- ⚠ • The chair **must not** be used by individuals with bedsores.
- ⚠ • You must not retrofit the chair with loose parts: for example, straps and belts.
- ⚠ • Make sure that children and other users do not get their hands and feet caught in the carrying straps.
- ⚠ • Protac SenSit® Nature may only be used by individuals with a maximum weight of 200 kg.
- ⚠ • Protac SenSit® Nature is not intended to be mobile, and under no circumstances should it be placed or affixed to a platform with wheels.
- Avoid smoking in the chair. Lost embers or cigarettes can cause burns.
- ⚠ • Never leave a turned on computer or other electrical devices in the chair as they generate heat which may cause the cover to deteriorate, or in the worst case cause the electrical unit to ignite as the heat cannot escape.
- **Must not** be used near an open flame.
- ⚠ • **Do not** leave a person sitting in the chair in direct sunlight for long periods of time.
- ⚠ • The user must use sunscreen if Protac SenSit® Nature is used in the sun.
- ⚠ • The seat of the chair **MUST** be checked for overheating if the chair has been in direct sunlight before placing a person in the seat in the sun.

Incidents

- Incidents related to the use of Protac Sensit® Nature must be reported to the manufacturer
- Serious incidents related to the use of Protac Sensit® Nature must be reported to the manufacturer and the competent authority of the country where the incident takes place.

Purpose and function of the product

All Protac products are developed from the theory of sensory integration. The theory originates in the USA and was developed by occupational therapist and psychologist A. Jean Ayres back in the 1970s. The theory is based on how the brain processes sensory impressions and how the different sensory stimuli can be influenced through weight and pressure on the tactile sense (sense of touch) in the skin and the proprioceptive sense (sense of movement) in muscles and joints. The pressure, movement and weight of the balls stimulate these particular senses.

GB

The Protac Sensit® Nature was developed as an aid for individuals with functional impairments. The Protac Sensit® Nature is suitable for children, youths, adults and the elderly.

- Brain damage and other neurological illnesses.
- ADHD and autism
- Psychiatric illnesses
- Developmental disorders, sensory disorders.
- Motor and mental restlessness.
- Concentration problems and learning difficulties.

The Protac SenSit® Nature chair is used to give relief or as a supplement for individuals with brain damage who are wheelchair users or bedridden. The chair's stable shape supports the neck, head and body, and the balls in the seat and back make it easy to adjust the chair and create a comfortable sitting and resting position.

Individuals with ADHD and autism can use the Protac SenSit® Nature chair as a 'contained space' where the user is surrounded by the balls and receive the sensory stimuli needed to ensure calm and a sense of security. The chair can be used to eliminate sensory impressions so that the user can concentrate on school homework for example.

Individuals with mental illness often suffer from motor and mental restlessness. The Protac SenSit® Nature chair can be used as a 'safe place' to ensure calm and a sense of security for the user. Weight and pressure of the balls also lead to better body awareness and improved well-being.

Protac SenSit® Nature is used for people with e.g. developmental disorders, sensory disorders, to create increased sensory stimulation for the body.

The Protac SenSit® Nature is used for people with physical and mental unrest to ensure calm and a sense of security as well as better body awareness.

Using the product

Protac Sensit® Nature can be used with the neck wings placed inside the chair, behind the user's back and with the side wings hanging down the side of the chair. This contains the body, without the user feeling the weight of the wings, which may be a good start.

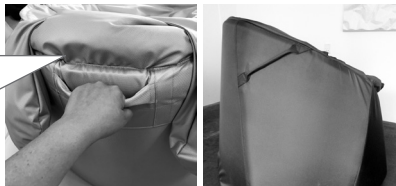
Protac Sensit® Nature can be used with the neck and/or side wings placed over the user so he/she can feel the weight and pressure of the balls. This leads to a better sense of the body's boundaries and body awareness.

When the user is wearing the neck wings, the care staff/helper may apply light pressure to the shoulder area to increase the sensory stimulation in the shoulders.

You can move Protac Sensit® Nature using carrying handles in the sides and at center back.

When using the Protac Sensit® Nature for individuals with for example spasticity, create a recess at the back of the seat by moving the balls so that the user can sit in a reclined position further back in the seat.

GB

Protac SenSit® Nature is delivered assembled and ready for use

Product lifetime

The product's expected lifetime is three years. The product's lifetime is based on correct use, cleaning, washing, service and maintenance. Products that are used in excess of the above lifetime must be assessed for future use. The user is responsible for continuously assessing whether the product can be used in excess of the lifetime stated.

Cleaning

The cover for Protac SenSit® Nature can be washed. The need and frequency of washing depend on the use of the chair. Follow the instructions below when washing and cleaning the chair.

- The cover can be washed in a washing machine, min. 7 kg, using a detergent without bleach and perfume. Fabric softener must not be used. See the washing instructions inside the cover.
- The seat cover can be unzipped and washed separately using a detergent without bleach and perfume. Fabric softener must not be used. See the washing instructions inside the cover.
- The ball bags in the neck and side wings can be washed in a washing machine, min. 7kg, using the supplied washing bags. Use detergent without bleach and perfume. Fabric softener must not be used. See the washing instructions inside the wings.

Dismantling instructions before wash:

1. Remove the innerbag from the neck support.
2. Remove the ball bags from side-/ neck wings. The ball bags must be washed separately. The included laundry bag must be used.
3. Zip off the seat cover.
4. Detach the inner bag, with balls and granulate, from the velcro in the back and take it out of the chair.
5. Unzip the chambers for the back and front-foam plates at the bottom.
6. Remove the front and back foam plates.
7. Close all zippers. Zip seat cover to the cover, before washing.

Assembly after wash:

1. Unzip the seat cover.
2. Unzip the front foam plate chamber.
3. Insert the front foam plate in the chamber, remember to turn the side with the plastic plate inward. Close the chamber with the zipper.
4. Unzip the chamber for the back foam plate.
5. Insert the back foam plate in the chamber, remember to turn the side with the plastic plate inward. Close the chamber with the zipper.
6. Place the innerbag inside the cover. Make sure the velcro pieces fit at the back
7. Zip on the seat cover. Attach the velcro on the seat cover to the velcro on the inner bag.
8. Insert the innerbag in the neck support, the ball bags for neck and side wings d in the chambers. Closes with zipper. Remember to attach the ball bags to the neck and side wings inside with velcro.

Find videofilm showing how to assemble the Protac SenSit® Nature at: www.protac.dk

Please note! This is a job for two people!

Before using any stain removers, do a test on a non-visible area (bottom of chair) to make sure that it does not damage or change the colour of the fabric. Protac does not recommend the use of stain removers.

- ⚠ • Do not wash the inner bag with balls and granulate. Can be wiped with a well-wrung cloth soaked in warm soapy water. Can be wiped off with alcohol.
- ⚠ • Do not wash the inner bag in the neck rest

Maintenance

Over time a recess will form in the seat as the granulate at the bottom 'settle' through use. It is therefore important to turn the chair upside down once in a while and shake it a little to allow the granulate to become evenly distributed again. This will restore the shape of the seat. Inspect the chair for defects and deficiencies prior to each use, and inspect the chair's seams, zips, fabric, straps, balls and general condition once a year. If serious defects or deficiencies are discovered, the Protac SenSit® Nature must not be used until the defects have been repaired and safety has been restored.

The Protac SenSit® Nature can be wiped with a well-wrung cloth soaked in warm soapy water. Can be wiped off with alcohol.

- ⚠ • Protac SenSit® Nature does not tolerate temperatures below 0 degrees celcius
- After heavy rain, turn the chair upside down and wipe the bottom with a dry cloth
- ⚠ • If the chair is placed directly in the sun, the included rain cover can be pulled over the chair, to protect it from the sun's rays when the chair is not in use. The fabric will fade, but it is postponed if the chair is protected with the raincover when not in use

Protac accepts no liability for damages resulting from incorrect washing and maintenance.

Transport and storage

It is easy to move the Protac SenSit® Nature chair using the two carrying straps fitted on each side of the chair.

- ⚠ To avoid strain on the back, it is recommended that two people lift the Protac SenSit® Nature, using the handles on each side.

During transport, the chair is packed in a cardboard box, which protects the chair against dirt and moisture.

When the chair is not in use, the supplied rain cover must be pulled over the chair. The rain cover protects the chair against, among other things, bird droppings, sun and very heavy rain.

Disposal

Protac SenSit® Nature can be divided into the following components for waste disposal purposes:
Plastic and fabric.

The plastic balls can be disposed of as hard plastic.

All parts of the chair can be disposed of as combustible waste.

Waste must be disposed of in accordance with applicable rules.

Warranty

There is a two-year warranty against manufacturer's defects.

GB

The warranty will be voided in the following cases:

- Overload of Protac SenSit® Nature
- Harsh use of Protac SenSit® Nature
- Lack of or incorrect maintenance of Protac SenSit® Nature

Accessories

- Protac SenSit® Nature Puff
- Protac SenSit® Carry bag

Protac SenSit® Nature Puff

Protac SenSit® Nature Puff is a foot/leg rest that may be used together with the Protac SenSit® Nature.

Use of the product

The Protac SenSit® Nature Puff can be used as a foot/leg rest, together with the Protac SenSit® Nature.

The Protac SenSit® Nature Puff can be used as a stool.

General precautions - Safety

-  • The Puff must not be used unsupervised by individuals who require special care.
-  • Must not be used by individuals with pressure ulcers on their calves and heels.
-  • The Puff must not be used as a stool, by individuals with bedsores.
-  • Take care not to trip over the footstool if the user is unsure of his/her balance.
-  • Protac SenSit® Nature Puff must not be placed or affixed to a platform with wheels.
-  • Avoid smoking when sitting on the stool. Lost embers or cigarettes can cause burns.
-  • Must not be used near an open flame.
-  • Do not leave a person sitting on the stool in direct sunlight for long periods of time.
-  • The user must use sunscreen if Protac SenSit® Nature Puff is used in the sun.
-  • Max. weight 150 kg
- The seat of the stool **MUST** be checked for overheating if the chair has been in direct sunlight before placing a person in the seat.

Cleaning

- The cover can be washed in a washing machine, min. 7 kg, using a detergent without bleach and perfume. Fabric softener must not be used. See the washing instructions inside the cover.
- 
- Do not wash the inner bag with balls and granulate. Can be wiped with a well-wrung cloth soaked in warm soapy water. Can be wiped off with alcohol.

Traceability: Production month, year and batch number are marked on the washing label. These must not be removed from the product.

Key to symbols

Example only. Always follow the recommended care instructions on the label found inside the product

Washing bowl: Maximum washing temperature (see care label)

Triangle:
Do not bleach

Square with a circle:
tumble drying possible
(see care label)

Iron:
Do not iron

Circle:

Warning:
See under Safety

User information

For outdoor use

CE marked according to
(EU) 2017/745 Class I
medical devices

Manufacturer

Batch number

Accessories

- Protac Sensit® Nature Puff
- Protac SenSit® Carry bag

Spare parts

- Cover for chair
- Foam plate for back
- Foam plate for front
- Innerbag with balls and granulate.
- Ballbags for neck- and side wings
- Innerbag for neckrest
- Raincover

Technical specifications

Dimensions:	Back height	Diameter	Weight
Protac SenSit® Nature	78 cm.	90 cm.	approx. 20 kg.
Protac SenSit® Nature	85 cm.	90 cm.	approx. 21 kg.

Patent: Protac SenSit® is patented.

Standards: Comply with general requirements for aids acc. to the following harmonized standards:

EN ISO 13485:2016/A11:2021: Medical Devices – Quality management systems

EN ISO 14971:2019/A11:2021: Medical devices - Application of risk management to medical devices

DS/EN ISO 21856:2022 : Medical devices - Assistive products - General requirements and test methods.

Watercolumn for outer fabric: 10.000 mm - EN ISO 811: 2018

Service information

Only the balls used by the manufacturer may be used in the Protac SenSit® Nature.

Guide

- to daily pre-use inspection of the Protac SenSit® Nature

- 1 • All parts are in place, no loose parts.
 (wings and neck pillow)

- 2 • Straps are not loose, seams are OK.

- 3 • Straps are not loose, seams are OK.

- 4 • Seat height is OK (no sinking of the seat).

- 5 • No sign of holes or damage to the fabric
 and seams

Protac A/S
Niels Bohrs Vej 31 D, Stilling
DK-8660 Skanderborg
Tel.: +45 86 19 41 03
e-mail: protac@protac.dk

Distributor list: Please refer to: www.protac.dk/kontakt

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Assembly after wash:

1. Unzip the seat cover.
2. Unzip the front foam plate chamber.
3. Insert the front foam plate in the chamber, remember to turn the side with the plastic plate inward. Close the chamber with the zipper.
4. Unzip the chamber for the back foam plate.
5. Insert the back foam plate in the chamber, remember to turn the side with the plastic plate inward. Close the chamber with the zipper.
6. Place the innerbag inside the cover. Make sure the velcro pieces fit at the back
7. Zip on the seat cover. Attach the velcro on the seat cover to the velcro on the inner bag.
8. Insert the innerbag in the neck support, the ball bags for neck and side wings in the chambers. Closes with zipper. Remember to attach the ball bags to the neck and side wings inside with velcro.

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During transport, the chair is packed in a cardboard box, which protects the chair against dirt and moisture. When the chair is not in use, the supplied rain cover must be pulled over the chair. The rain cover protects the chair against, among other things, bird droppings, sun and very heavy rain.

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-  • The Puff must not be used unsupervised by individuals who require special care.
-  • Must not be used by individuals with pressure ulcers on their calves and heels.
-  • The Puff must not be used as a stool, by individuals with bedsores.
-  • Take care not to trip over the footstool if the user is unsure of his/her balance.
-  • Protac SenSit® Nature Puff must not be placed or affixed to a platform with wheels.
-  • Avoid smoking when sitting on the stool. Lost embers or cigarettes can cause burns.
-  • Must not be used near an open flame.
-  • Do not leave a person sitting on the stool in direct sunlight for long periods of time.
-  • The user must use sunscreen if Protac SenSit® Nature Puff is used in the sun.
-  • Max. weight 150 kg

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