## Protac SenSit® Straight

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GB - Directions for use

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SE - Bruksanvisning

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## Protac SenSit® Straight

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# GB - Directions for use

## Protac SenSit® Straight

You have chosen a Protac quality product. Protac guarantees that you have purchased an original Danish designed product developed in partnership with professionals.

Since 1994, Protac has developed, manufactured and sold sensory stimulating aids to the healthcare sectors in Denmark, Europe, Australia and New Zealand.

Protac's products are developed from the theory of sensory integration and are well documented with respect to research, documentation and case descriptions. Protac's production process focuses on environment and safety. Protac plastic balls are certified STANDARD 100 by OEKO-TEX® nr. 2076-310 DTI.

For more details, go to [www.protac.dk](http://www.protac.dk)

GB

## Labelling

The products are labeled in accordance with Regulation (EU) 2017/745 Class I medical devices. CE- and care labels with information about the manufacturer, raw materials and washing instructions are placed inside the cover.

Symbol explanation on label: see the Key to symbols section

## General precautions - Safety

- ⚠ • Please pay attention to the reaction of the user, if he or she does not like being encased in the chair's neck and side wings.
- ⚠ • Make sure that the user does not overheat if he/she is embraced by the chair's neck and side wings.
- ⚠ • Users with poor breathing must use the side and neck wings with care and never unsupervised.
- ⚠ • The chair must not be used unsupervised with individuals who require special care.
- ⚠ • Check to see whether any balls might be irritating the user. This is particularly important if the user is incapable of independent movement.
- ⚠ • The chair must not be used by individuals with bedsores.
- ⚠ • Make sure that children and other users do not get their hands and feet caught in the handle.
- ⚠ • Protac SenSit® Straight is not designed for outdoor use.
- ⚠ • Protac SenSit® Straight may only be used by individuals with a maximum weight of 150 kg.
- ⚠ • The Protac SenSit® Straight is not intended to be mobile, and under no circumstances should it be placed or affixed to a platform with wheels.
- ⚠ • Avoid smoking in the chair. Lost embers or cigarettes can cause burns.
- ⚠ • Never leave a turned on computer or other electrical devices in the chair as they generate heat which may cause the cover to deteriorate, or in the worst case cause the electrical unit to ignite as the heat cannot escape.
- ⚠ • Must not be used near an open flame.

## Purpose and function of the product

All Protac's products are developed from the theory of sensory integration. The theory originates in the USA and was developed by occupational therapist and psychologist A. Jean Ayres back in the 1970s. The theory is based on how the brain processes sensory impressions and how the different sensory stimuli can be influenced through weight and pressure on the tactile sense (sense of touch) in the skin and the proprioceptive sense (sense of movement) in muscles and joints. The pressure, movement and weight of the balls stimulate these particular senses.

The Protac SenSit® Straight was developed as an aid for individuals with functional impairments. The Protac SenSit® Straight is suitable for the elderly, adults, young people and children:

GB

- Brain damage and other neurological illnesses.
- ADHD and autism
- Psychiatric illnesses
- Developmental disorders, sensory disorders.
- Physical and mental unrest.
- Concentration problems and learning disorders.

The Protac SenSit® Straight is used for the elderly in care homes, who suffer from e.g. dementia. The chair's stable foam shape makes it safe to sit in and supports the neck, head and body. The balls inside of the seat and back make it easy to adjust the chair and create a comfortable sitting and resting position. The chair's seat height makes it very easy for users to sit down and get back up out of the chair.

The Protac SenSit® Straight is used to give relief or as a supplement for individuals with brain damage who are wheelchair users or bedridden. The chair's stable shape supports the neck, head and body, and the balls in the seat and back make it easy to adjust the chair and create a comfortable sitting and resting position.

Individuals with ADHD and autism can use the Protac SenSit® Straight as a 'contained space' where the user is surrounded by the balls and receive the sensory stimuli needed to ensure calm and a sense of security. The chair can be used to eliminate sensory impressions so that the user can concentrate on school homework for example.

Individuals with mental illness often suffer from physical and mental unrest. The Protac SenSit® Straight can be used as a 'safe place' to ensure calm and a sense of security for the user. The weight and pressure of the balls also leads to better body awareness and improved well-being.

Protac SenSit® Straight is used for people with e.g. developmental disorders, sensory disorders, to create increased sensory stimulation for the body.

The Protac SenSit® Straight is used for people with physical and mental unrest to ensure calm and a sense of security as well as better body awareness.

## Using the product

The Protac SenSit® Straight can be used with the neck wings placed inside the chair, behind the user's back and with the side wings hanging down the side of the chair. This contains the body, without the user feeling the weight of the wings, which may be a good start.

The Protac SenSit® Straight can be used with the neck and/or side wings placed over the user so he/she can feel the weight and pressure of the balls. This leads to a better sense of the body's boundaries and body awareness.

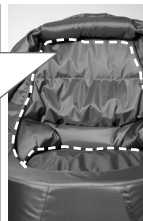
GB

When the user is wearing the neck wings, the care staff/helper may apply light pressure to the shoulder area to increase the sensory stimuli.

The back pillow is attached only with Velcro under the neck pillow. The back pillow is not attached to the sides or bottom, which makes the back pillow flexible and easy to adjust for the back and lumbar area to accommodate the individual user.

The seat cushion is reversible and has two different sides: one with sensory stimulating balls and one with pressure relieving foam.

Sensory stimulating balls

Reversible

Pressure relieving foam

The chair's foam bottom under the seat cushion and 20 cm up the sides are lined with incontinence material to protect the chair's foam from liquid (refer to label).

The Protac SenSit® Straight can be moved by using the handle center back.

**The Protac SenSit® Straight is delivered fully assembled**

## Dismantling instructions before wash:

1. Remove the innerbag from the neck support. Do not wash!
2. Remove the ball bags from the side-/neck wings. The ball bags must be washed separately. Use the enclosed laundrybag
3. Remove the seat cushion from the chair, unzip the cover with plastic balls and the incontinence cover.
4. Remove the 3 inner bags from the back pillow (one with plastic balls and two with polystyrene granulate). NB! Do not wash the two inner bags with polypropylen granulate!
5. Unzip and take off the chair cover.
6. All zippers must be zipped up before washing.

## Assembly after wash:

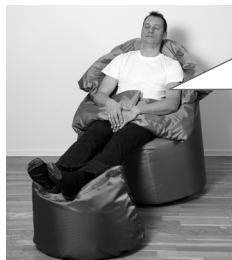
1. Put the cover on the chair and zip it up.
2. Place the inner bag of polystyrene balls in the neck support.
3. Insert the ball bags in the neck - and side wings. Close the zipper. DON'T FORGET to attach the ball bags to the neck- and side wings inside with velcro.
4. The seat cushion is fitted with incontinence cover and fabric cover with plastic balls. Close with a zipper. Place the seat in the chair.
5. Place the three inner bags inside of the back pillow's three compartments (one with plastic balls, two with polypropylen granulate). IMPORTANT! Place the bag with polypropylen granulate in the top compartment. Place the bag of plastic balls in the middle compartment and the other bag with polypropylen granulate in the bottom compartment.
6. Affix the back pillow under the neck cushion with Velcro.

## Accessories

Protac SenSit® Puff

The Protac SenSit® Puff can be used as a foot/leg rest, together with the Protac SenSit® Straight.

### Using the accessory

The Protac SenSit® Puff can be used as a foot/leg rest, together with the Protac SenSit® Straight.

The Protac SenSit® Puff can be used as a stool.

## General precautions - Safety

- ⚠ • Must not be used unsupervised with individuals who require special care.
- ⚠ • Must not be used by individuals with pressure ulcers on their calves and heels
- ⚠ • Take care not to trip over the footstool if the user is unsure of his/her balance.

- ⚠ • Do not use to sit on if you have decubitur.
- ⚠ • Do not place on a plate with wheels.
- ⚠ • Avoid smoking when sitting on the Puff.
- ⚠ • Must not be used near an open fire.
- ⚠ • May only be used as a stool by individuals with a maximum weight of 150 kg.

## Product lifetime

The product's expected lifetime is five years.

The product's lifetime is based on correct use, cleaning, washing, service and maintenance.

Products that are used in excess of the above lifetime must be assessed for future use. The user is responsible for continuously assessing whether the product can be used in excess of the lifetime stated.

GB

## Cleaning

It is recommended to clean and wash the covers as the Protac SenSit® Straight chair may become dirty over time. The need and frequency of washing depend on the use of the chair. Follow the instructions below when washing and cleaning the chair.

- The outer fabric can be washed in a regular washing machine, min. 7 kg, using a detergent without bleach. See also the washing instructions inside the cover.
- The seat cover with plastic balls and incontinence cover can be removed and washed separately using a detergent without bleach. See also the washing instructions inside the cover.
- All inner bags with plastic balls can be washed in a regular washing machine, min. 7 kg, using the supplied washing bags and a detergent without bleach. See also the washing instructions inside the wings.
- The back pillow cover can also be washed. See also the washing instructions inside the back pillow.

Before using any stain removers, do a test on a non-visible area (bottom of chair) to make sure that it does not damage or change the colour of the fabric. Protac does not recommend the use of stain removers.

Protac accepts no liability for damages resulting from incorrect washing and maintenance.

- ⚠ • All inner bags with polypropylene granulate must not be washed

## Maintenance

Inspect the chair for defects and deficiencies prior to each use, and inspect the chair's seams, zips, fabric, handles, balls and general condition once a year. If defects or deficiencies are discovered, the Protac SenSit® Straight chair must not be used until the defects have been repaired and safety has been restored.

The Protac SenSit® Straight can be wiped with a well-wrung cloth soaked in warm soapy water. Can be wiped off with alcohol

## Transport and storage

It is easy to move the Protac SenSit® Straight chair using the carrying handle on the back side of the chair.

- ⚠ • The chair should preferably be lifted by two people

-  • When moving the chair, do not lift but pull on the handle as this puts less strain on the back. However, if moving the chair over a distance of more than two metres, the chair should be lifted to prevent unnecessary wear to the fabric.

During transport, the chair is packed in a special polypropylene bag or export carton, which protects the chair against dirt and moisture. The chair must be stored in a safe place when not in use, preferably covered to protect it against dirt and moisture.

## Disposal

The Protac SenSit® Straight can be divided into the following components for waste disposal purposes: Plastic, foam and fabric.

The plastic balls can be disposed of as hard plastic.

All parts of the chair can be disposed of as combustible waste.

Waste must be disposed of in accordance with applicable rules.

GB

## Warranty

There is a two-year warranty against manufacturer's defects.

### The warranty will be voided in the following cases:

- Overload of the Protac SenSit® Straight
- Harsh use of the Protac SenSit® Straight
- Lack of or incorrect maintenance of the Protac SenSit® Straight

**Traceability:** Production month, year and batch number are marked on the washing label.

These must not be removed from the product.

## Key to symbols

Example only. Always follow the recommended care instructions on the label found inside the product

Washing bowl: Maximum washing temperature (see care label)

Warning:  
See under Safety

Triangle:  
Do not bleach

User information

Square with a circle:  
tumble drying possible  
(see care label)

For indoor use

Iron:  
Do not iron

Manufacturer

Circle:  
Do not dry clean

Batch number

## Spare parts

- Cover for chair
- Seat cover with balls
- Incontinence cover for the seat
- Back pillow with balls and granulates
- Ball bags for neck- and side wings
- Bag with granulates for neck support

## Technical specifications

Dimensions:

Protac SenSit® Straight:                    height: 100 cm    diameter: 90 cm    weight: approx. 18 kg

GB

**Patent:** The Protac SenSit® Straight is patented.

**Standards:** Comply with general requirements for aids acc. to the following harmonized standards:

**Clinical evaluation:** EN 12182:2012

**Risk analysis:** DS/EN ISO 14971

**Crash test:** EN 1022:2005 and EN 16139:2013

**Fire test:** EN 1021-part 1 & 2: 2014

## Service information

Only the balls used by the manufacturer may be used in the Protac SenSit® Straight.

## Guide

- to daily pre-use inspection of the Protac SenSit® Straight

- 1    • All parts are in place, no loose parts. (wings, neck and back pillows and seat cushion)
- 2    • Handle is not loose, seams are OK
- 3    • All zippers are closed
- 4    • No sign of holes or damage to the fabric and seams

Protac A/S  
Niels Bohrs Vej 31 D, Stilling  
DK-8660 Skanderborg  
Tel.: +45 86 19 41 03  
e-mail: protac@protac.dk

Distributor list: Please refer to: [www.protac.dk/kontakt](http://www.protac.dk/kontakt)

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