
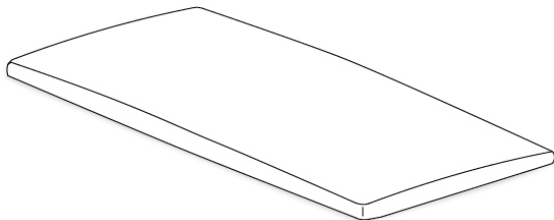
# PROTAC®

INTEGRATING SENSES

## Incontinence covers

- DA Brugsvejledning
- EN Operations manual
- DE Gebrauchsanweisung
- FI Käyttöohje
- FR Mode d'emploi
- NL Gebruiksaanwijzing
- SV Bruksanvisning
- ES Instrucciones de uso
- IT Istruzioni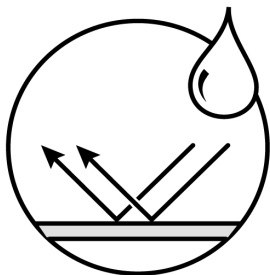

|                 |    |
|-----------------|----|
| Dansk .....     | 4  |
| English.....    | 7  |
| Deutsch .....   | 10 |
| Suomi .....     | 13 |
| Français .....  | 16 |
| Nederlands..... | 19 |
| Svenska .....   | 22 |
| Español .....   | 25 |
| Italiano.....   | 28 |

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# English

|                                      |   |
|--------------------------------------|---|
| Traceability .....                   | 7 |
| Key to symbols - product label ..... | 7 |
| Safety - general precautions .....   | 8 |
| Use of the product .....             | 8 |
| Use environment.....                 | 9 |
| Maintenance .....                    | 9 |
| Cleaning .....                       | 9 |
| Key to symbols - care label .....    | 9 |
| Service life and warranty .....      | 9 |
| Disposal .....                       | 9 |

EN

You have chosen a Protac quality product. Protac guarantees that this product is of original Danish design and developed in consultation with experts. Since 1994, Protac has developed, manufactured and sold sensory-stimulating aids to the healthcare sector in Denmark. Today, Protac products are sold worldwide.

Further information can be found at [www.protac.dk](http://www.protac.dk).

## Traceability

Production month, year and batch number can be found on the product label. Do not remove the product label from the product.

## Key to symbols - product label

|                                                                                     |                           |                                                                                     |                       |
|-------------------------------------------------------------------------------------|---------------------------|-------------------------------------------------------------------------------------|-----------------------|
|  | Instructions for use      |  | Article number        |
|  | Lot number                |  | For indoor use        |
|  | Manufacturer              |  | Production month/year |
|  | Warning: See under Safety |                                                                                     |                       |


## Safety - general precautions

|                                                                                   |                                                                                      |
|-----------------------------------------------------------------------------------|--------------------------------------------------------------------------------------|
|  | Not for unsupervised use by users unable to care for themselves                      |
|  | Incontinence covers can cause strangulation and must be used with caution.           |
|  | Avoid smoking next to the product. Dropped embers or cigarettes can cause burns.     |
|  | Never leave switched on electrical appliances that emit heat on or near the product. |
|  | Do not use near open fire.                                                           |

## Use of the product


The incontinence cover is to be used as a regular duvet cover on the blanket or as protection for the top mattress in case of an incontinent user.

The incontinence cover is easy to slip on as the zipper opens over 2 sides.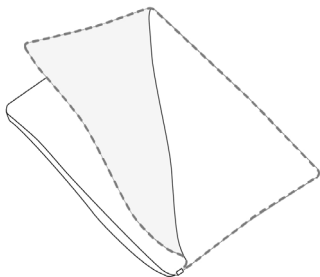

When using with a blanket, put a regular duvet cover on top of the incontinence cover. Make sure that the opening of the duvet cover is opposite the opening of the incontinence cover.

This is not applicable for incontinence covers with mesh.

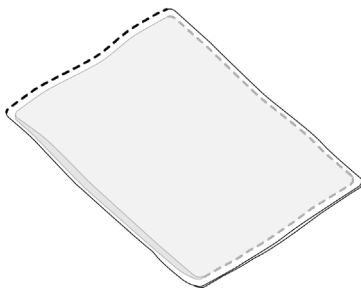When using with a top mattress, make sure to put a sheet on top.

## Use environment

Incontinence covers are for indoor use. They can be used both in institutions and private homes.

## Maintenance

It is recommended to wash the incontinence cover on a regular basis.

## Cleaning

The incontinence cover can be washed in a regular household washing machine. The frequency of washing depends on use. When washing, please follow the instructions on the care label. Washing instructions can also be found on page 31 of this manual.

EN

Protac does not recommend the use of stain remover, as it will damage the fabric.

## Key to symbols - care label

|                                                                                   |                             |                                                                                   |                       |
|-----------------------------------------------------------------------------------|-----------------------------|-----------------------------------------------------------------------------------|-----------------------|
|  | Maximum washing temperature |  | Line dry in the shade |
|  | Do not wash                 |  | Do not iron           |
|  | Can be tumble dried         |  | Do not dryclean       |
|  | Do not tumble dry           |  | Do not bleach         |

## Service life and warranty

The expected service life of the product is 5 years. This is based on correct use, cleaning, washing and maintenance.

Products used beyond the expected service life must regularly undergo a maintenance inspection which implies checking for wear and tear in fabric and seams.

The product comes with a 2-year warranty, see page 2.

The warranty will be invalid in case of:

- overload or excessive use of the product
- lack of or incorrect maintenance of the product

## Disposal

Incontinence covers must be disposed of as combustible waste according to national regulations.

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Protac A/S