

PROTAC®

INTEGRATING SENSES

Protac® KneedMe

Protac® KneedMe To Go

- DA Brugsvejledning
- EN Operations manual
- DE Gebrauchsanweisung
- FI Käyttöohje
- FR Mode d'emploi
- NL Gebruiksaanwijzing
- SV Bruksanvisning
- ES Instrucciones de uso
- IT Istruzioni

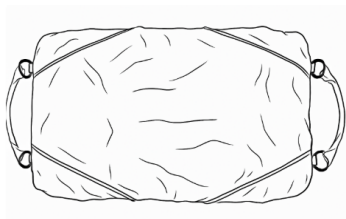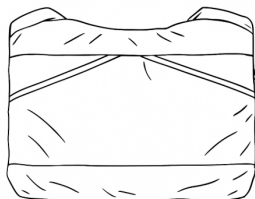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English

Marking	10
Incidents - adverse events	10
Traceability	10
Key to symbols - product label	10
Safety - general precautions	10
Use of the product	11
Use environment	11
Indications	11
Contra-indications	11
Cleaning	12
Key to symbols - care label	12
Service life and warranty	12
Disposal	12

You have chosen a Protac quality product. Protac guarantees that this product is of original Danish design and developed in consultation with experts.

Since 1994, Protac has developed, manufactured and sold sensory-stimulating aids to the healthcare sector in Denmark.

Today, Protac products are sold worldwide.

All Protac products have been developed on the basis of the theories of sensory integration, and they are supported by extensive research, documentation and case studies.

The theory of sensory integration by A. Jean Ayres forms the basis of the development of all Protac products. The theory is based on how the brain processes sensory impression and how stimulation of the muscle-joint-sense and the deep tactile sense has a calming and organising effect on the sensory system.

All Protac products contain balls, and it is the movement of the balls that provide the dynamic deep-point pressure on the body and stimulate the receptors in the deep tactile sense and the muscle-joint sense. This enhances the body awareness, alleviates sensory disorder and has a calming effect on both body and mind.

Further information can be found at www.protac.dk.

Marking

The products are CE-marked in accordance with Regulation (EU) 2017/745 on medical devices.

EU declaration of conformity can be found at www.protac.dk.

Incidents - adverse events

- Incidents related to the use of Protac® KneedMe or Protac® KneedMe To Go must be reported to Protac.
- Adverse events related to the use of Protac® KneedMe or Protac® KneedMe To Go must be reported to the National Competent Authority and to Protac.

EN

Traceability

Production month, year and batch number can be found on the product label.

Do not remove the product label from the product.

Key to symbols - product label

	Instructions for use		Maximum user weight
	Medical device		Article number
	Lot number		For indoor use
	Manufacturer		Production month/year
	CE marked in accordance with Regulation (EU) 2017/745 on medical devices		
	Warning: See under Safety		

Safety - general precautions

	The shoulder strap on Protac® KneedMe To Go is not to be used as a restraining device, neither in a chair, nor in a car, nor anywhere else.
	The knee blanket is not to be used by the driver of a motorized vehicle.
	Avoid smoking next to the product. Dropped embers or cigarettes can cause burns.
	Never leave switched on electrical appliances that emit heat on or near the product.
	Do not use near open fire.
	Do not use other balls in the product than those supplied by the manufacturer.

Use of the product

Protac Protac® KneedMe and Protac® KneedMe To Go are supplied ready for use. They are suitable for adults and for children above the age of 6.

The knee blanket is placed on the thighs and knees of a seated user.

Protac® KneedMe has pockets in which objects that the user finds comfortable to touch can be placed.

EN


Protac® KneedMe To Go is easy to take on the go.

The knee blanket can be folded and carried using the 2 handles or the shoulder strap.

The shoulder strap can be placed behind the user's lower back to ensure that the knee blanket doesn't slide off during use.

Protac® KneedMe To Go has pockets that can be used for a tablet.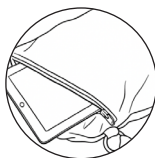

Use environment

Protac® KneedMe and Protac® KneedMe To Go can be used both in institutions and private homes.

Indications

Both knee blankets are sensory-stimulating aids intended to alleviate difficulties concentrating and restlessness.

Both knee blankets contain balls which move and apply dynamic pressure on to the body, thereby stimulating the muscle-joint sense and the tactile sense. This increases body awareness and calms the brain and body, and the product thereby alleviates physical and psychological unrest.

Contra-indications

Persons unable to care for themselves are not to use the knee blankets unattended. The knee blankets are not to be used by persons with decubitus or persons at risk of developing decubitus.

Cleaning

Protac® KneedMe and Protac® KneedMe To Go can be wiped over with a cloth wrung out in soapy water. The cover and the ball bags can be washed separately in a regular household washing machine, note that the ball bag is held in place with buttons inside the cover. The frequency of washing depends on use. When washing, please follow the instructions on the care label.

Washing instructions can also be found on page 42-43 of this manual.

Protac does not recommend the use of stain remover.

EN

Key to symbols - care label

	Maximum washing temperature		Do not wash
	Can be tumble dried		Do not tumble dry
	Dry flat		Do not iron
	Line dry in the shade		Do not dryclean
			Do not bleach

Service life and warranty

The expected service life of the product is 5 years. This is based on correct use, cleaning, washing and maintenance.

Products used beyond the expected service life must regularly undergo a maintenance inspection which implies checking for wear and tear in fabric and seams.

The product comes with a 2-year warranty, see page 2.

The warranty will be invalid in case of:

- overload or excessive use of the product
- lack of or incorrect maintenance of the product

Disposal

The product can be broken down into the following components for disposal:

- The plastic balls can be disposed of as hard plastic.
- The fabric can be disposed of as textiles.

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Protac A/S