
PROTAC®

INTEGRATING SENSES

Protac MyFit®

- DA Brugsvejledning
- EN Operations manual
- DE Gebrauchsanweisung
- FI Käyttöohje
- FR Mode d'emploi
- NL Gebruiksaanwijzing
- SV Bruksanvisning
- ES Instrucciones de uso
- IT Istruzioni

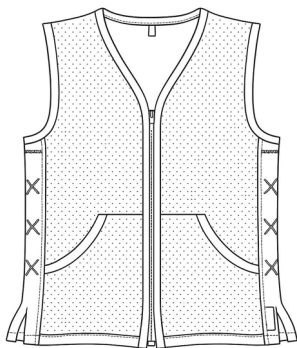Dansk	5
English.....	9
Deutsch.....	13
Suomi	17
Français	21
Nederlands.....	25
Svenska	29
Español	33
Italiano.....	37

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English

Marking	10
Incidents - adverse events	10
Traceability	10
Key to symbols - product label	10
Safety - general precautions	10
Use of the product	11
Use environment	11
Indications	11
Contra-indications	11
Cleaning	12
Key to symbols - care label	12
Service life and warranty	12
Disposal	12

You have chosen a Protac quality product. Protac guarantees that this product is of original Danish design and developed in consultation with experts.

Since 1994, Protac has developed, manufactured and sold sensory-stimulating aids to the healthcare sector in Denmark.

Today, Protac products are sold worldwide.

All Protac products have been developed on the basis of the theories of sensory integration, and they are supported by extensive research, documentation and case studies.

The theory of sensory integration by A. Jean Ayres forms the basis of the development of all Protac products. The theory is based on how the brain processes sensory impression and how stimulation of the muscle-joint-sense and the deep tactile sense has a calming and organising effect on the sensory system.

All Protac products contain balls, and it is the movement of the balls that provide the dynamic deep-point pressure on the body and stimulate the receptors in the deep tactile sense and the muscle-joint sense. This enhances the body awareness, alleviates sensory disorder and has a calming effect on both body and mind.

Further information can be found at www.protac.dk.

Marking

The products are CE-marked in accordance with Regulation (EU) 2017/745 on medical devices.

EU declaration of conformity can be found at www.protac.dk

Incidents - adverse events

- Incidents related to the use of Protac MyFit® must be reported to Protac.
- Adverse events related to the use of Protac MyFit® must be reported to the National Competent Authority and to Protac.

Traceability

Production month, year and batch number can be found on the product label.

Do not remove the product label from the product.

Key to symbols - product label

	Instructions for use		Maximum user weight
	Medical device		Article number
	Lot number		For indoor use
	Manufacturer		Production month/year
	CE marked in accordance with Regulation (EU) 2017/745 on medical devices		
	Warning: See under Safety		

Safety - general precautions

	Do not use as a swim vest or life jacket.
	Do not use together with a torso seat belt.
	Remove the vest before performing the Heimlich maneuver.
	Avoid smoking next to the product. Dropped embers or cigarettes can cause burns.
	Never leave switched on electrical appliances that emit heat on or near the product.
	Do not use near open fire.
	Do not use other balls in the product than those supplied by the manufacturer.

Use of the product

Protac MyFit® is supplied ready for use.

Put on the vest and close the zipper.

EN

The vest can be adjusted in both sides at in the back by pulling or loosening the elastic string. As a precaution, the elastic strings are placed in a concealed system.

After adjustment, the loose piece of elastic string can be rolled together and placed in the small pocket.

Use environment

Protac MyFit® can be used both in institutions and private homes.

Indications

Protac MyFit® is intended to alleviate physical and mental unrest.

Protac MyFit® is a sensory-stimulating vest which provides an enveloping feeling of the body. The balls in the vest stimulate the muscle-joint sense and the tactile sense through deep and dynamic touch-pressure and the vest thereby promotes an improved body awareness and calms the brain and body. This promotes concentration and increases the ability to learn.

Contra-indications

People unable to take care of themselves or people with poor balance should not wear the vest unattended.

Cleaning

Protac MyFit® can be washed in a regular household washing machine. The frequency of cleaning depends on frequency of use. When washing, please follow the instructions on the care label which can also be found on page 42 of this manual.

Protac does not recommend the use of stain remover.

Key to symbols - care label

	Maximum washing temperature		Do not wash
	Can be tumble dried		Do not tumble dry
	Dry flat		Do not iron
	Line dry in the shade		Do not dryclean
			Do not bleach

Service life and warranty

The expected service life of the product is 5 years. This is based on correct use, cleaning, washing and maintenance.

Products used beyond the expected service life must regularly undergo a maintenance inspection which implies checking for wear and tear in fabric and seams.

The product comes with a 2-year warranty, see page 2.

The warranty will be invalid in case of:

- overload or excessive use of the product
- lack of or incorrect maintenance of the product

Disposal

The product can be broken down into the following components for disposal:

- The plastic balls can be disposed of as hard plastic.
- The fabric can be disposed of as textiles.

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Protac A/S