
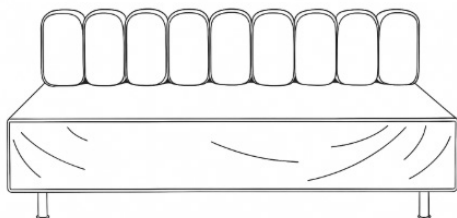
PROTAC®

INTEGRATING SENSES

Protac® SensCircle

- DA Brugsvejledning
- EN Operations manual
- DE Gebrauchsanweisung
- FI Käyttöohje
- FR Mode d'emploi
- NL Gebruiksaanwijzing
- SV Bruksanvisning
- ES Instrucciones de uso
- IT Istruzioni

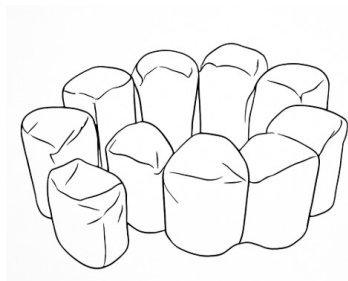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English

Marking	10
Incidents - adverse events	10
Traceability	10
Key to symbols - product label	10
Safety - general precautions	10
Use of the product	11
Use environment	11
Indications	11
Contra-indications	11
Cleaning	12
Key to symbols - care label	12
Service life and warranty	12
Disposal	12

EN

You have chosen a Protac quality product. Protac guarantees that this product is of original Danish design and developed in consultation with experts.

Since 1994, Protac has developed, manufactured and sold sensory-stimulating aids to the healthcare sector in Denmark.

Today, Protac products are sold worldwide.

All Protac products have been developed on the basis of the theories of sensory integration, and they are supported by extensive research, documentation and case studies.

The theory of sensory integration by A. Jean Ayres forms the basis of the development of all Protac products. The theory is based on how the brain processes sensory impression and how stimulation of the muscle-joint-sense and the deep tactile sense has a calming and organising effect on the sensory system.

All Protac products contain balls, and it is the movement of the balls that provide the dynamic deep-point pressure on the body and stimulate the receptors in the deep tactile sense and the muscle-joint sense. This enhances the body awareness, alleviates sensory disorder and has a calming effect on both body and mind.

Further information can be found at www.protac.dk.

Marking

The products are CE-marked in accordance with Regulation (EU) 2017/745 on medical devices.

EU declaration of conformity can be found at www.protac.dk

Incidents - adverse events

- Incidents related to the use of Protac® SensCircle must be reported to Protac.
- Adverse events related to the use of Protac® SensCircle must be reported to the National Competent Authority and to Protac.

Traceability

Production month, year and batch number can be found on the product label.

Do not remove the product label from the product.

Key to symbols - product label

	Instructions for use		Maximum user weight
	Medical device		Article number
	Lot number		For indoor use
	Manufacturer		Production month/year
	CE marked in accordance with Regulation (EU) 2017/745 on medical devices		
	Warning: See under Safety		

Safety - general precautions

	Avoid smoking next to the product. Dropped embers or cigarettes can cause burns.
	Never leave switched on electrical appliances that emit heat on or near the product.
	Do not use near open fire.
	Do not use other balls in the product than those supplied by the manufacturer.

Use of the product

Protac® SensCircle is supplied ready for use.

The cushion can be used in many different ways. It can be used as demarcation of the body, both in the seated position and in the lying position.

EN

The cushion consists of a cover and three inner modules, one containing balls and the other two containing granulate. The three inner modules are interchangeable.

The cushion can be folded as a chair or be used to lie on.

Use environment

Protac® SensCircle is for indoor use. The cushion can be used both in institutions and private homes.

Indications

Protac® SensCircle is a sensory-stimulating aid intended to enclose and delimit the body, thereby promoting an improved body awareness.

Protac® SensCircle contains balls which move and apply dynamic pressure on to the body, thereby stimulating the muscle-joint sense and the tactile sense. This increases body awareness and calms the brain and body, and the product thereby alleviates physical and psychological unrest.

Contra-indications

Protac® SensCircle is not to be used by persons with decubitus or persons at risk of developing decubitus.

Protac® SensCircle is not to be used unattended by persons unable to take care of themselves.

Cleaning

Both the cover and the inner modules can be wiped over with a cloth wrung out in soapy water. They can also be disinfected with anti-septic wipes. The frequency of washing depends on use.

Remove the inner modules from the cover and close the zipper. All parts can be washed in a washing machine, please follow the instructions on the care label. Washing instructions can also be found on page 41-43 of this manual.

Protac does not recommend the use of stain remover.

Key to symbols - care label

	Maximum washing temperature		Do not wash
	Can be tumble dried		Do not tumble dry
	Dry flat		Do not iron
	Line dry in the shade		Do not dryclean
			Do not bleach

Service life and warranty

The expected service life of the product is 5 years. This is based on correct use, cleaning, washing and maintenance.

Products used beyond the expected service life must regularly undergo a maintenance inspection which implies checking for wear and tear in fabric and seams.

The product comes with a 2-year warranty, see page 2.

The warranty will be invalid in case of:

- overload or excessive use of the product
- lack of or incorrect maintenance of the product

Disposal

The product can be broken down into the following components for disposal:

- The plastic balls can be disposed of as hard plastic.
- The granulate can be disposed of as combustible waste.
- The fabric can be disposed of as textiles.

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Protac A/S