PROTAC®

INTEGRATING SENSES

Protac SenSit® Frame

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EN Operations manual

Dansk.....	5
English.....	8

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English

EN

Traceability	8
Key to symbols - product label	8
Incidents - adverse events	8
Safety - general precautions	9
Use of the product	9
Maintenance	10
Cleaning	10
Transport and storage	10
Service life and warranty	10
Disposal	10

You have chosen a Protac quality product. Protac guarantees that this product is of original Danish design and developed in consultation with experts. Since 1994, Protac has developed, manufactured and sold sensory-stimulating aids to the healthcare sector in Denmark. Today, Protac products are sold worldwide.

Further information can be found at www.protac.dk.

Traceability

Production month, year and batch number can be found on the product label. Do not remove the product label from the product.

Key to symbols - product label

	Instructions for use		Article number
	Lot number		For indoor use
	Manufacturer		Production month/year
	Warning: See under Safety		

Incidents - adverse events

Protac SenSit® Frame is an accessory to Protac SenSit® chair. Incidents concerning the use of Protac SenSit® Frame together with a Protac SenSit® chair must be reported to Protac. Adverse events must be reported to the National Competent Authority and to Protac.

Safety - general precautions

	Not for unsupervised use by users unable to care for themselves
	Avoid smoking next to the product. Dropped embers or cigarettes can cause burns.
	Never leave switched on electrical appliances that emit heat on or near the product.
	Do not use near open fire.
	To be used only with Protac SenSit® chair.
	It is important to place the chair correctly in the frame.
	The wheels must be locked before a user sits in the chair.
	Do not move the frame while a user is sitting in the chair.

Use of the product

Protac SenSit® Frame must be assembled before use. Assembly instruction can be found on pages 3-4.

Protac SenSit® Frame is for indoor use only.

It is important to place the chair in the frame as shown on the photo (next page) to benefit from the functionality of the frame.

- Combines a comfortable seating position with the sensory stimulating effect of the chair.
- Makes it easier to get in and out of the chair
- Enables flexible use

Protac SenSit® Frame is stable with a slightly tilted plate at the bottom, which has several user friendly functions. When used with Protac SenSit® chair, the handles on the frame makes it easier for the user to get in and out of the chair. At the same time, the frame makes the chair a bit higher because of the tilted bottom plate, and this makes the chair more accessible for people with functional impairments.

The frame also makes it possible to move the chair around, e.g. in a common room, and thereby making the use of the chair more flexible.

Maintenance

It is recommended to wipe off the frame on a regular basis.

Cleaning

Protac® MyRest can be washed in a regular household washing machine. The frequency of washing depends on use.

Protac SenSit® Frame can be cleaned with a soft cloth wrung out in ordinary soap water. Frequency depends on use. The frame can be disinfected with antiseptic wipes.

Transport and storage

The frame is equipped with 2 wheels at the back which make it possible to tilt the frame and drive it around without having to carry it. This facilitates moving the frame around and also cleaning underneath it.

Service life and warranty

The expected service life of the product is 5 years. This is based on correct use, cleaning, washing and maintenance.

Products used beyond the expected service life must regularly undergo a maintenance inspection which implies checking for wear and tear and loose screws.

The product comes with a 2-year warranty, see page 2.

The warranty will be invalid in case of:

- overload or excessive use of the product
- lack of or incorrect maintenance of the product

Disposal

Metal frame and wooden plate to be disposed of according to national legislation.

DA	Besøg vores hjemmeside for at finde din lokale forhandler.
EN	Visit our website to find your local dealer.
DE	Besuchen Sie unsere Website, und finden Sie einen Händler in Ihrer Nähe.
FI	Vieraile verkkosivullamme löytääksesi paikallinen jälleenmyyjäsi.
FR	Rendez-vous sur notre site Internet pour trouver les coordonnées du revendeur local.
NL	Breng een bezoek aan onze website om te zien waar u onze producten kunt kopen.
SV	Besök vår webbplats för att hitta en lokal återförsäljare.
ES	Visite nuestro sitio Web para localizar su distribuidor local.
IT	Visita il nostro sito web per trovare il rivenditore di zona

www.protac.dk/uk/contact

Protac A/S
Niels Bohrs Vej 31 D, Stilling
DK-8660 Skanderborg
Danmark
Tel.: +45 86 19 41 03
e-mail: protac@protac.dk